ACRYCUBE

Ruthinium

PMMA CAD-CAM CUBE

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 Fabbricante

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(GB) PMMA BLOCK FOR CAD-CAM MACHINING. USED FOR THE MANUFACTURING OF PROVISIONAL ANATOMICAL BRIDGES AND/OR CROWNS. DESTINED TO OVERCOME MASTICATORY MALFUNCTIONS.

INDICATIONS

Block made of cross-linked high-molecular-weight resin, which guarantees faithful reproduction of details and resistance even at thin thicknesses. Can also be used for the fabrication of castable pieces.

DEVICE CLASSIFICATION

Class I, Rule 5 according to Annex VIII Regulation (EU) 2017/745

Class I, Rule 5 according to Annex 9 The Medical Devices (Amendment) (Great Britain) Regulations 2023.

TARGET POPULATION

The PMMA blocks are intended for all age groups and genders. Patients are individuals with masticatory dysfunction who require interventions to restore mastication through the clinical application of anatomical bridges and crowns.

USER PROFILE (PROFESSIONALS/PROFESSIONALS)

The target population for the PMMA blocks are dental professionals and technicians with specific skills in digital dentistry, CAD-CAM technology and anatomical bridge and crown fabrication.

INDICATIONS FOR USEUSED

for the fabrication of anatomical bridges and crowns. The Block is made from high molecular weight cross-linked resins that faithfully reproduce details while maintaining strength even for thin thicknesses. It can also be used for the fabrication of castable parts.

CLINICAL BENEFITS

The PMMA Block for CAD-CAM processing is suitable for patients in need of dental restorations due to compromised tooth structures, fractures or trauma. It is specifically intended for the CAD-CAM manufacture of anatomical bridges and crowns.

CONTRAINDICATIONS:

- Not indicated in the case of a proven allergy to the components of Acry Cube Ruthinium. - All uses not indicated.

AVAILABLE FORMATS:

UNIVERSAL ATTACHMENT

SIZE SMALL 15,4x38x19,4 LARGE 15,4x48x19,4

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MAIN FEATURES:

- Medical device intended for dental CAD/CAM: the dental laboratory provides milling to obtain temporary teeth, modelled structures such as crowns, temporary bridges. - High stability.
- Chromatic inalterability.

INSTRUCTIONS FOR USE:

1. The material must be processed in an environment with controlled conditions to avoid contamination of the material. In case of contamination by grease or oil, clean the material with alcohol.
2. Vacuum the chips during machining, avoid overheating the material to prevent deformation.
3. Finish as usual, with burs, rubbers, brushes and with pumice. Polish to a high gloss using brushes and polishing products (Acry Lux Crema, Acry Lux Liquido, Acry Lux Pasta), avoiding excessive overheating.

DESIGN SPECIFICATIONS:

- Minimum crown thickness : 3.5 mm occlusal and 0.6 mm circumferential; - Offset : 3.0 mm;
- Distance to outer edge: 2.0 mm;
- Bridge connector radius : between 1.5 mm and 2.0 mm;
- Bridge connector cross-section: between 7.0 mm² 12.6 mm²

PERMITTED APPLICATIONS:

- Single elements : maximum 3 for molars, maximum 4 for incisors; - Front and rear crowns;
- Front and rear bridges;

WARNINGS:

- Before using the artefact in the oral cavity, clean and/or as defined by the doctor/dentist.
- The product is not autoclavable;
- Avoid cleaning with steam;
- All commercially available eugenol-free cements or adhesives can be used for cementation.

STORAGE CONDITIONS: We recommend keeping the product in a cool, dry place. Acry Cube Ruthinium is not a hazardous product. Disposal must be in accordance with the laws of the competent authorities. The waste can be treated as urban waste or recycled as normal plastic. The product is designed for professional use in the dental sector. Please observe the instructions in the instructions for use. The manufacturer accepts no liability for damage resulting from incorrect or improper use. The user is obliged to personally check the suitability of the product for the intended uses, especially if these uses are not listed in the instructions for use. Do not use after the expiry date.



Warning: The user is required to report any accident and/or serious incident that may occur in relation to the device to both the manufacturer and the competent authority of the Member State in which the user and/or patient is established. Instruct the patient with appropriate cleaning protocols.

RECOMMENDED WORKING STRATEGY

TOOL TYPE							MILLING PARAMETERS				
Ø TOOL mm	Ø d mm	Nr. of Blades	R mm	l mm	i mm	L mm	RPM (S)	TOOL FEED XY (F)	TOOL FEED Z (F)	Ae (XY)	Ap (Z)
2	2	2	1	1.6	20	50	18500	1200	800	1	0.5
1	1	2	0.5	0.8	16	50	25000	1000	600	0.15	0.08
0,6	0,6	2	0.3	0,5	10	50	30000	500	500	0.05	0.05

ØD = Constrained by gripper measure of the milling machine

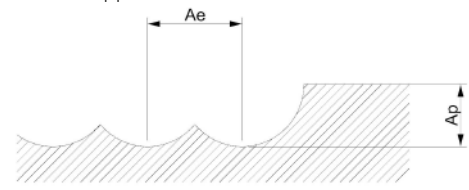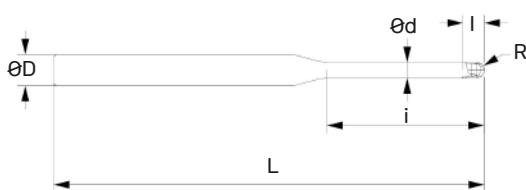TOOL TYPE and MILLING PARAMETERS = recommended values and working strategy, consult the tool supplier.

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Classe I, Regra 5 de acordo com o Anexo VIII Regulamento (UE) 2017/745

Classe I, Regra 5 de acordo com o Anexo 9 The Medical Devices (Amendment) (Great Britain) Regulations 2023.

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