Stargel

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(GB) DUPLICATING GEL FOR CHROME-COBALT INVESTMENT.

INSTRUCTIONS: Fill the melting tank with the duplicating gel cut into small pieces. Set the melt temperature to 85°C and the pouring temperature to 48-50°C. Alternatively, heat the gel in a double boiler, mixing it continuously to help it melt.

Model preparation: Immerse the model in warm water (40-45°C) for at least 15 minutes, then dry it and place it on the base of the duplicating flask.

Duplicating: Pour the duplicating gel into the flask. Leave to cool for at least 10 minutes at room temperature, then immerse ¾ of the flask in cold water for another 15-20 minutes (in order to speed up the time it takes to cool). Open the flask and remove the model. Carefully dry the duplication and immediately pour in the investment material.

Reusing the duplicating gel: Wash the gel carefully and, if it isn't reused straight away, store it in an airtight container before melting it again. After being reused a number of times, if the gel becomes too dense, add a small amount of water before melting it again.

IMPORTANT: Never mix new duplicating gel with duplicating gel that has already been used.

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Ruthinium® - Dental Manufacturing S.p.A.